medi day gel

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GB: The selected active ingredients are suitable for caring for sensitive skin and relieving tension. Massage medi day gently into your skin every day before getting dressed. The revitalising skin care promotes the wearing comfort of mediven compression stockings.

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GB: Please apply medi night after doffing mediven compression stockings. The ingredient witch hazel soothes your skin and helps it regenerate. medi night and medi day are the ideal combination for your legs.

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GB: The cream foam is suitable for drier skin and is very well tolerated. You can put on your compression stockings immediately after applying medi soft because the foam is absorbed rapidly and completely. It gives topbands additional hold.

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GB: The spray you can spray directly onto and through your stocking. You can feel the refreshing effect of menthol straight away. You can use medi fresh several times a day as and when required to supplement the skin care products.

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GB: The moisturising aloe vera gel is eminently suited for caring for residual limbs and the skin. It is absorbed residue-free within 1–2 minutes. Selected ingredients care for your skin and prevent skin irritation. Apply sparingly twice a day.

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GB: Thanks to its valuable ingredients the Special cream is particularly kind to the skin and reduces irritation on the residual limb, preventively and in acute cases. Use the cream sparingly and only on affected local areas. It is very economic in use!

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GB: Just a few drops of the mild Neutral soap enable thorough, residue-free cleaning of the residual limb and medi Liners and stockings. The added collagen and aloe vera care for your skin and support skin regeneration.

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GB: Donning and doffing Liners without a textile cover or lubricating coating is particularly comfortable when you use medi Spray: spray the outside of the Liner before donning and ease it down gently. The Liner slides down easily and simply.

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GB: Care for your compression stocking is just as vital for wearing comfort as the good fit. medi clean cleans gently and is formulated to match the yarn and colours. The fragrance-free hand wash liquid is dermatologically tolerated.

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GB: For the removal of shoe marks before the daily stocking wash.

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GB: Gentle to your skin and holds your compression stockings in place.

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